

# Warranty Information

Your Lifemax product is guaranteed for one year against defective parts or construction. This guarantee is from the date of purchase and is only valid with proof of purchase, Please note this guarantee does not cover normal wear and tear or damage caused by accident or maltreatment. The extended rights of this guarantee are in addition to the consumer statutory rights and do not limit the effect of these statutory rights.

Please register your product at <https://www.lifemaxuk.co.uk/warranty>.

# Technical Specification

Model No	331	Battery Type	2 x AAA
Alarms	1	Size	93 X 70 X 15mm



This is a WEEE symbol. It indicates that when the end-user discards this product, it must be sent to separate collection facilities for recovery and recycling. By separating this product from other household-type waste, the volume of waste sent to incinerators or land-fills will be reduced and natural resources will thus be conserved. Lifemax Ltd provide an end-of-life solution for this product when sold in the UK by accepting back any end of life items for processing by a local registered waste disposal company.



Lifemax Limited, 1 Pipers Court,  
Thatcham, Berkshire, RG19 4ER

Lifemax (Ireland), 51 Bracken Road,  
Sandyford, Dublin, D18 CV48

[www.lifemaxuk.co.uk](http://www.lifemaxuk.co.uk)



Made in China

Lifemax

# Under Pillow Vibration Alarm Clock

SKU: 331

## Instruction Manual



Thank you for purchasing a Lifemax Under Pillow Vibration Alarm Clock. We hope you'll get many years of use from this product. We recommend you read this document thoroughly to familiarise yourself with how your product works, how you should use it safely and effectively and how to look after your product to ensure it continues working as it should.


# Product Information

Ideal for those with hearing loss or for shift workers, this under pillow alarm clock will vibrate to wake even the deepest sleeper.

## What's included...?

Under Pillow Vibration Alarm Clock  
Travel Case

## How to use...

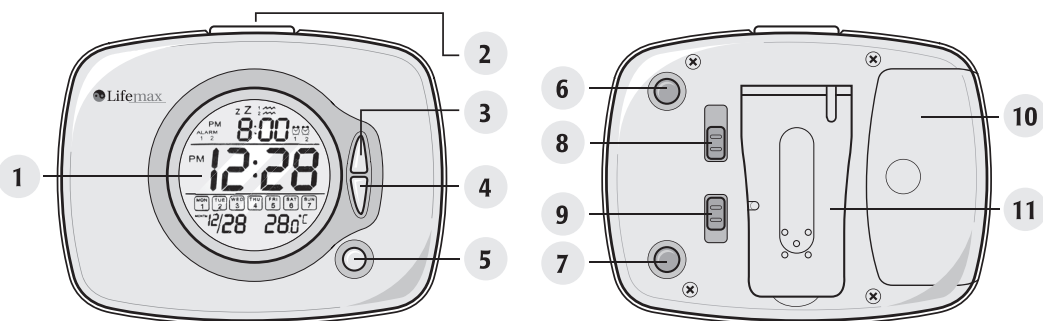

1. Display
2. Snooze/Light
3. Up
4. Down
5. Alarm On/Off
6. Alarm Set
7. Clock Cal Set
8. Buzzer/Vibration
9. Key Lock
10. Battery Compartment
11. Stand

### SETTING THE TIME

Press and hold the CLOCK CAL SET button for 2 seconds to enter time set mode and the minute will flash on the LCD screen. Use the UP/DOWN buttons to adjust the minutes. Press the CLOCK CAL SET button to advance to the hour setting and use the UP/DOWN buttons to adjust. Press the CLOCK CAL SET button again to advance to the year setting and use the UP/DOWN buttons to adjust. Press the CLOCK CAL SET button again to advance to the month setting and use the UP/DOWN buttons to adjust. Press the CLOCK CAL SET button

again to advance to day of the month setting and use the UP/DOWN button to adjust. Press the CLOCK CAL SET button again to confirm and complete the time setting.

### SETTING THE ALARM

Press and hold the ALARM SET button for 2 seconds to enter the alarm set mode and the hour will flash on the upper section of the LCD screen. *[Please note the hour setting can be in either 12 or 24 hour mode depending on how you are viewing the alarm on the home screen. If in 12 hour mode you should see AM or PM to the far left of the upper section of the LCD screen. This can be changed by using the UP button prior to setting the alarm. Use the UP/DOWN buttons to adjust the hours.]* Press the ALARM SET button again to advance to the minute setting and use the UP/DOWN buttons to adjust. Press the ALARM SET button again to confirm the alarm setting and to return to the normal display mode. You can use the ALARM ON/OFF button on the front of the alarm to activate or deactivate the alarm. On the back of the alarm you can use the alarm BUZZER/VIBRATION selector switch to choose buzzer and vibration, just buzzer or just vibration.

### ALARM SNOOZE FUNCTION

When the alarm sounds you can press the SNOOZE button to snooze the alarm for around 8 minutes.

### ACTIVATE DISPLAY BACKLIGHT

Press LIGHT button activate the display backlight.

### ACTIVATE KEY LOCK FUNCTION

To lock the UP, DOWN, CLOCK CAL SET and ALARM SET keys on the alarm move the KEY LOCK selector switch to the locked position.

### PLEASE NOTE

IF YOU TURN THE CLOCK OVER YOU WILL SEE A RECTANGULAR SWITCH WITH A LOCK/ UNLOCK SYMBOL. IF THIS IS SWITCHED DOWN IT WILL LOCK ALL BUTTONS. MAKE SURE THIS IS SWITCHED UP WHEN SETTING YOUR CLOCK.

## Safety Information

This product uses 2 x AAA batteries (not included). Use only designated battery type. Do not use re-chargeable batteries. Do not mix battery types. Read instructions carefully and install batteries correctly observing the polarity (+-) sign to avoid leakage. Always replace sets of batteries at the same time. Do not mix old and new batteries. Do not attempt to recharge non-rechargeable batteries. Remove batteries from the product when it will not be used for an extended period of time to avoid leakage. Remove exhausted batteries to avoid leakage. Do not short out battery contacts. Do not dispose of batteries in fire. Do not immerse electrically operated products in water or other liquid. Wipe clean only.