

INSTRUCTION MANUAL



**Safer Walking
GPS Device**

Model: RAVT01



Thank you for purchasing a Safer Walking GPS Device.
Please read the instructions carefully and keep for future reference.

Contents

Package contents.....	Pg3
Device Layout	Pg3
What is GPS Locating Device	Pg3
Setting Up and Getting Started	Pg4
Using Safer Waking App	Pg5
- Menu	Pg6
- History	Pg6
- Zones	Pg6
- Events	Pg7
- Setup.....	Pg7
SOS Button and Two Way Voice Call	Pg8
Battery Life and Charging	Pg8
Low Battery.....	Pg9

Package Contents

1 x GPS Locating Device	1 x Mains Adaptor
1 x SIM card already fitted	1 x Instruction Manual
1 x Charging Doc	
1 x USB Charging Cable	

Device layout



What is GPS Locating device?

The locating device is an aid that can give people who may get lost and confused the independence to walk outside safely.

This water resistant GPS Locating device is compact and could be attached to a keyring or lanyard and easily fits in a pocket.

There is an SOS button to raise an alarm and two way voice call function.

Setting Up and Getting Started


1. Put your GPS locator on charge for 3 – 4 hours
2. Go to <http://saferwalkingactivation.com> and click on 'Register' and fill in your details to register your account.



Account Registration

Account Registration form with fields for Name, Email, Password, Confirm Password, Product Package (Tracker With SIM - £14.99), Number of Devices (1), and Seller. Includes a 'Subscribe Now' button and a 'Pay with card' section with fields for Card Number, Expiration Date, and CVV. Logos for VISA, Mastercard, American Express, and Discover are visible.

3. On your smartphone download the 'Safer Walking' app from the app store or google play store.

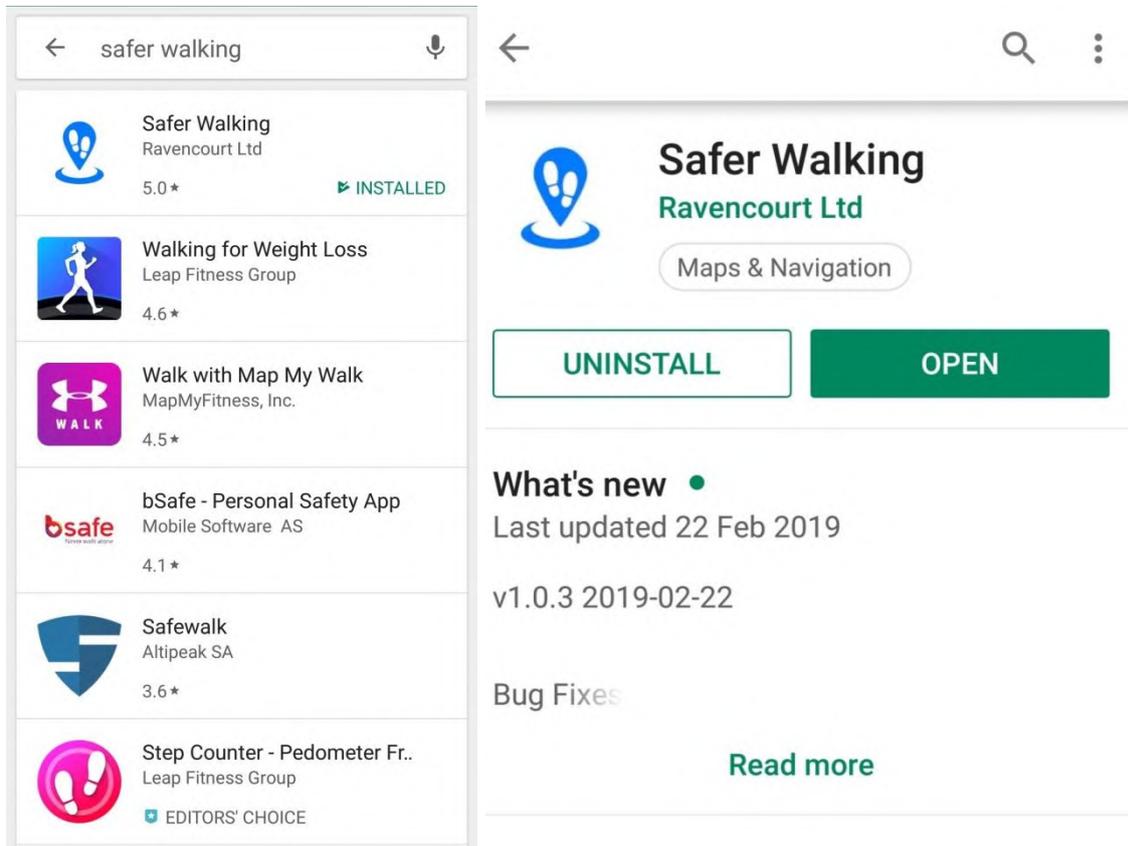


4. Open Safer Walking on your smartphone and log in with the email and password that you set up at <https://saferwalkingactivation.com>
5. Click get started now and give your device a name then enter the device ID and Phone number which is provided on a sticker on the box and also on the back of the safer walking device.
6. You will now see a map and next to your device name it might say “Your device will appear once a connection is made”.

You may need to walk around outside with the safer walking device for it to establish a GPS connection. This may take 5 – 10 minutes to establish a connection for the first time. If this doesn't establish a connection please check that the device ID is entered correctly in 'Setup'.

Using Safer Waking App

GPS Location

A GPS locating device uses global positioning satellites to show it's location on a map via the safer walking mobile app. The location updates every 3 minutes when moving and is accurate to around 10M when outside. Please note that information does not update properly while the person is driving in the car on a train or bus or in a building.

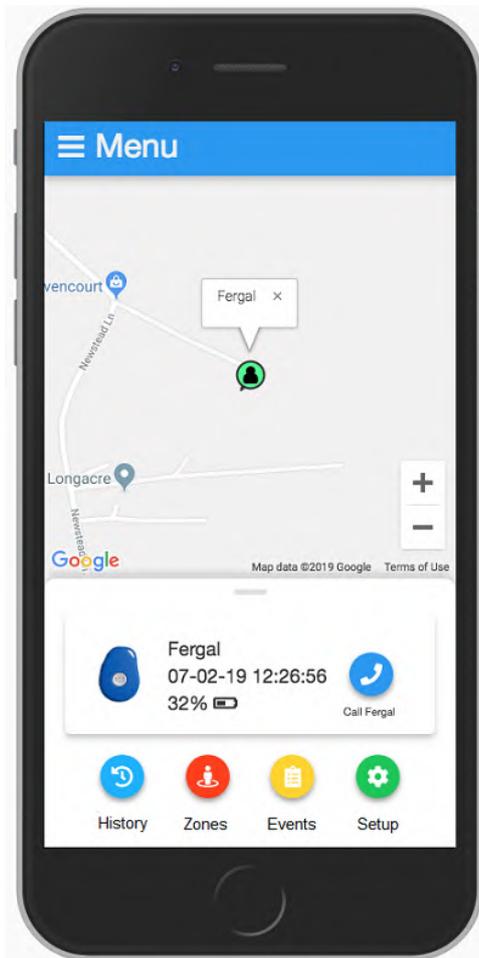


Menu

- Home – Taking you to main screen
- Add Device – You can add additional device by completing fields “Device Name”, “Device ID” and “Device Phone number” and press Add button. Please note you can add up to 5 devices on one account at www.saferwalkingactivation.com, each device will be charged at £14.99 per month.
- Settings:
 - o *Account Settings* – you can update personal information (Username/Email/Password)
 - o *Tracker Settings* – you can update device information (Device Name/Device ID/Device Phone number) or remove device
- Logout – log out from Safer Waking App account

History

The safer walking app will store the history of your device’s locations for one month.

When you click on the history icon you can choose a date and time “From” and a date and time “To”. Pressing “View History” will then show you a red line on the map of the journey that the locating device has taken with an arrow to indicate direction.



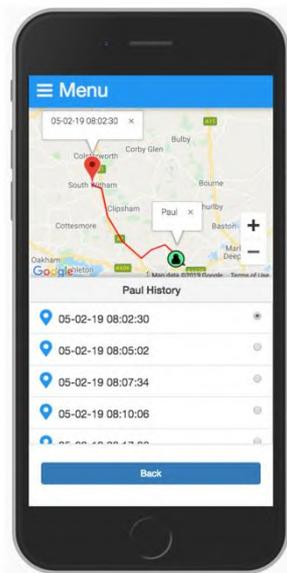


Zones

The safer walking mobile app allows you to set ‘zones’. For example Home, Work, Activity centre, Surgery.

You will get a sound notification to your smart phone every time the device enters/exits the zone. These Zones are sometimes referred to as Geo fences.

You can set zones and receive notifications each time your locating device exits or enters these zones. To do this press the zones icon and then ‘add zone’ you can zoom in and out by moving two fingers together or apart and move the map to move the zone. You can adjust the radius of the zone using the slider. Finally give your zone a name and press ‘add’.

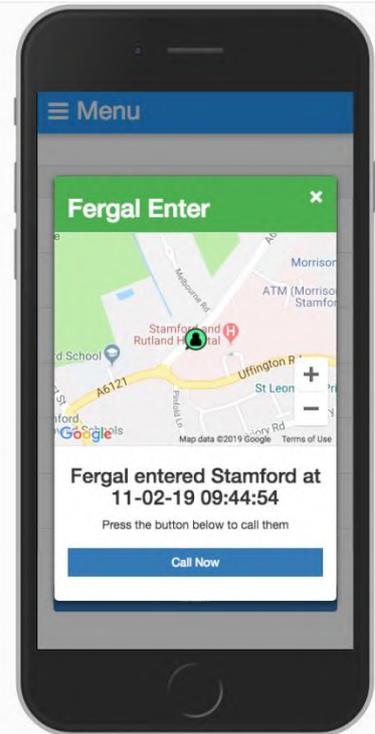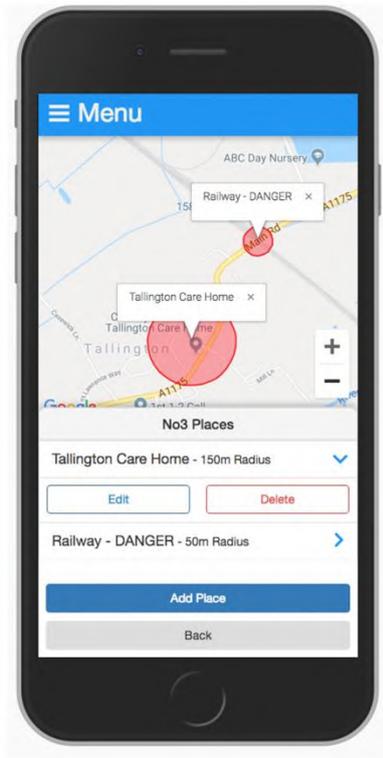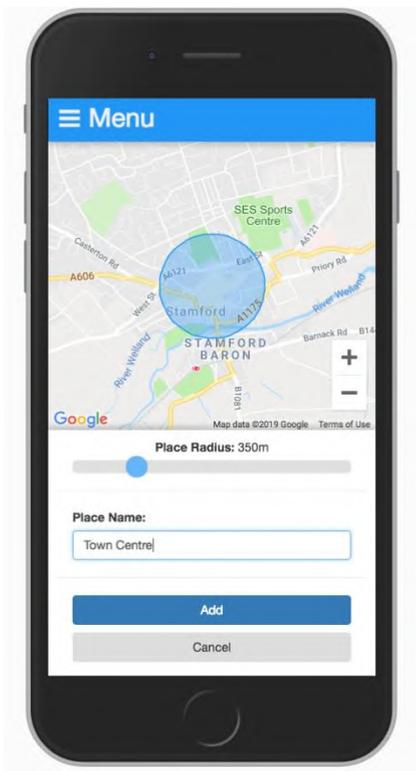

Events

Pressing the 'events' icon will give you a list of passed notification in chronological order.


Setup

Pressing the setup icon allows you to check and edit the device id, phone number and device name.

SOS Button and Two Way Voice Call

The GPS locating device has an SOS button. It is a good idea to test the SOS alarm first.

When SOS button is pressed and held for 3 seconds it will vibrate and 'beep' within 1 minute this will send a unique notification to your smart phone inviting you to make a call to the device.

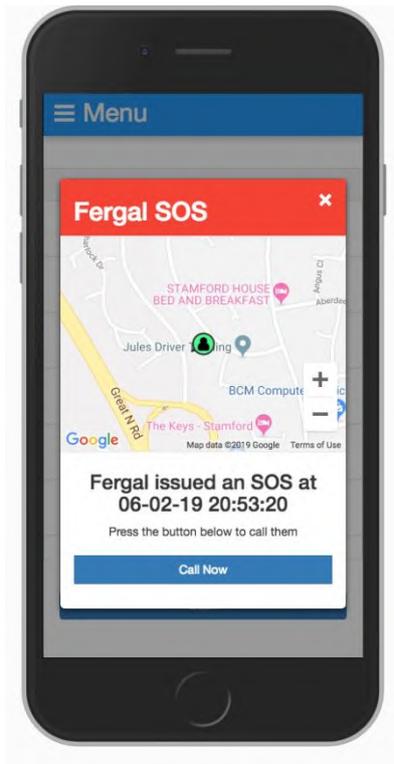

Notification will have a unique sound so please make sure that you have sound turned on and notifications enabled

When you open the notification you will be asked to make a call back to the locating device. The locating device cannot call out but you can have a two way voice call by making a call to the device.

The locating device will auto answer but first it rings to indicate to the user that a call is incoming.

Please note that you can make a call to Safer Walking device at any time using the "Call" button on Safer Walking App Home screen.

Battery Life and Charging

The approximate battery life is displayed on the safer walking app. You will also receive low batter notifications through the app. This allows you to make a call to the device to remind the person carrying the device to charge.

Normally battery lasts up to 48 hours since last full charge

The safer walking locating device can be charged easily just by resting in the charging dock or by USB to micro USB plugged directly into the locating device.

Low battery

A Low battery alert will come through when the device gets to around 20% battery.

The battery percentage is just an indication and not an accurate account of how long the battery might last. It's a good idea to call and remind the user to charge their locating device as a low battery notification come through.

FAQ's

How many people can download the app on each account?

5 people can download the app on the same account £2.99 per person per month.

Is there any additional monthly cost?

No £14.99 is the total you will need to pay each month.

How much do additional location devices cost to the same account?

Each GPS locating device that you add to your account will cost £14.99.

How far back is the history stored?

1 month.

How many zones can I set?

There is no limit on the amount of zones that can be set.

Can I track previous history dating back to before registering?

No, history will start to be stored after you have registered.

The logo for Ravencourt LIVING. It features a stylized purple raven head icon to the left of the word "Ravencourt" in a dark purple, sans-serif font. Below "Ravencourt" is the word "LIVING" in a large, teal, serif font.

www.ravencourtliving.com

www.saferwalkingactivation.com

info@ravencourtliving.com

01780 489100

If you no longer need your locating device

When the time comes that you no longer need your GPS location device please return your device to us free of charge by using the address FREEPOST Ravencourt. Please put a note inside stating the telephone number of your device and the device ID and DONATION. We can then donate your device to somebody who requires this product but may not be able to afford it.