



AQUA-CLINIC LEGATO FILTER CHANGE INSTRUCTIONS



Items required:


- One new filter
- One small adjustable spanner;
- One tube of silicon leak seal.

Changing the Activated Carbon Filter



Before changing the filter, switch off the hot and cold water supply.

To replace the cartridge, unscrew the front panel.

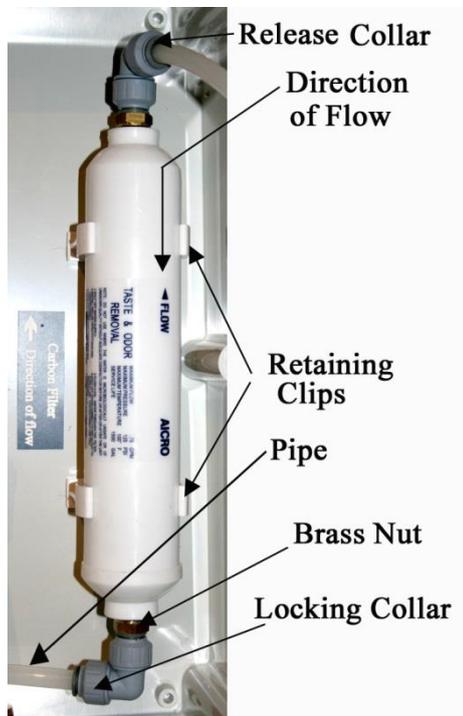

Then you need to release the nylon pipes from either end of the filter. Procedure:

1. Unscrew the Locking Collar to push in the Release Collar and pull out the pipe at both ends.
2. Remove the filter from the Retaining Clips.
3. Unscrew both Brass Nuts.
4. Clean off any residue clear sealant around the threads.
5. Apply a small amount of silicon leak sealer to the threads of the brass nuts.
6. Now screw the brass nuts into the filter. **BE GENTLE. DO NOT OVERTIGHTEN SO AS NOT TO DAMAGE THE THREADS ON THE FILTER**
7. Insert the filter into the retaining clips making sure that the direction of flow on the filter is the same as the filter label plate.
8. Push the nylon pipes back into the fittings.
9. Pull them back to make sure they don't come out.
10. Tighten the Collars.
11. Discard the old filter.

12. With the Variable flow control **CLOSED**, gentle switch your water supply back on and check for leaks.
13. Replace lid
14. Finally connect a water line to the outlet and run the pre-warm cycle.

BE CAREFUL: there will now be air in the system, so open the variable flow valve extremely carefully holding on to the water line or you will get wet.

As with any new carbon filter, the water coming out will be black. Run the water for a few minutes until it goes clear. Carbon filters need to be changed, on the average, every three to four months or after 100 to 150 treatments, whichever earlier. The best practice is to keep a count of your treatments and/or put a label on your filters with the filter change date.

IF YOU ARE STILL NOT SURE HOW TO CHANGE THE FILTER, ASK FOR PROFESSIONAL HELP!