

VisionAid Keypad

Quick Reference Guide

Controls



1. Top left corner square button: press to Toggle Document Mode / Virtual X/Y Mode

2. Top centre hexagonal button: press to Change Viewing Mode (Visualisation)

3./4. Top right triangular buttons: Press to Zoom in/Zoom out

5. Left side directly below 1 (square button) is a diagonal button: press for Quick capture

The centre of the keypad has a round button together with a larger inner and smaller outer ring of 8 navigation arrows outlined from the round button as follows:

6. Centre round button: press to Play / Pause

7./8. Next and previous Character

9./10. Next and previous Word

11./12. Next and previous Sentence

13./14. Next and previous Paragraph

15./16. Below button 13 are buttons to the left and right: Press for Next and Previous screen

17. In the centre below 15 & 16 square button: press to Append

18. Bottom centre rectangular button: press to Capture

Connecting and driver installation

Connect the VisionAid Keypad USB cable to your computer system. The cable is already connected to the top of the keypad; find a suitable USB socket on your computer and insert the USB connector. Take care to plug the USB connection in the correct way round. The keypad's driver will be installed automatically.

Depending on the version of Windows you are running, you may require an internet connection in order for the driver to be installed correctly.

Contact information

If you have any questions or queries with the operation of your VisionAid Keypad, please contact the local distributor that you purchased it from.

Designed and Manufactured by



VisionAid International Ltd

Bridge Lodge,
Spalding Common
Spalding, Lincolnshire
PE11 3AU
United Kingdom

Telephone:

+44 (0)1775 711 977

Email:

info@visionaid.com