

# Getting Started

# Prodig<sup>™</sup> DUO



DUO

DESKTOP

TABLET




# Humanware<sup>™</sup>

see things. differently.





## GETTING STARTED

1. Pull the Base out of its box by holding the temporary cardboard handle, and place it on a flat and level surface.

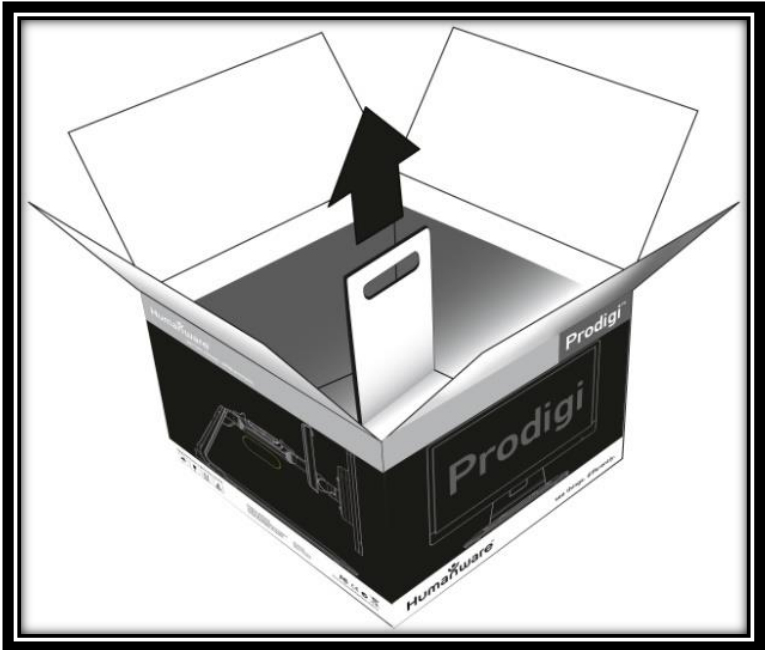


2. Make sure the capture and front panel surface is clean before placing the overlay stickers on your front panel, then carefully place the stickers at their appropriate location based on the images below:

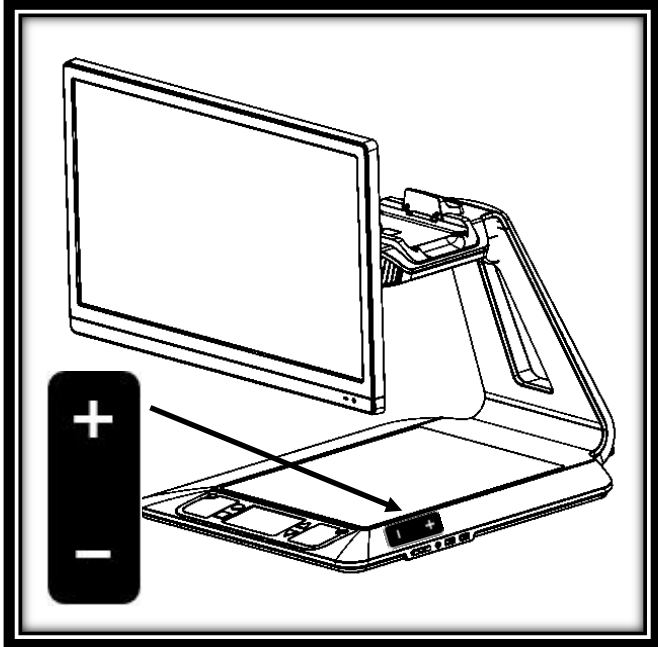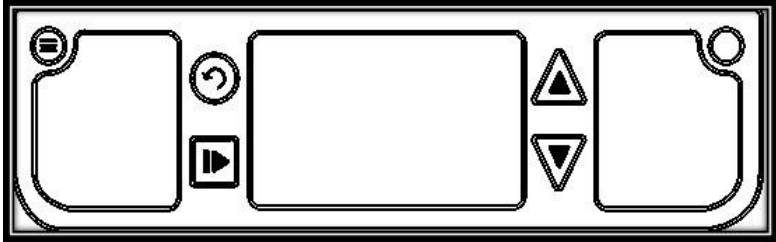

Once you have placed the stickers, wait 10 minutes to make sure they are well in place.



5. Place the Tablet on the Docking Station, screen side up and top edge facing towards the back. Push the two latches inward to connect the Tablet and power on the Base. Prodigy will take about 1 minute to initialize. HumanWare's logo will appear on the screen, followed by Prodigy's startup screen.

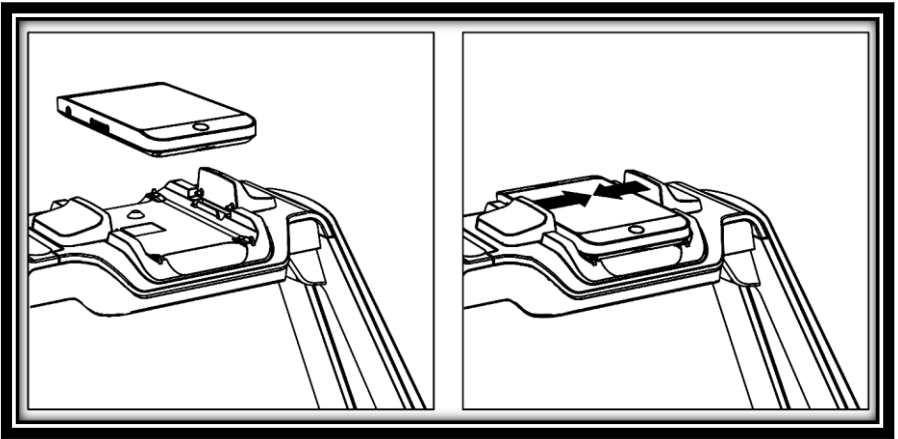

**Note:** If the Tablet's battery is drained, it may take a few minutes of recharge for Prodigy to initialize. Let the Tablet recharge initially on the docking station for **8 hours** to fully recharge the battery.



## Customer support

For customer support, please contact the HumanWare office nearest you or visit our Website at:  
[www.humanware.com/prodigi\\_support](http://www.humanware.com/prodigi_support)

**Global:** [support@humanware.com](mailto:support@humanware.com)

**North America:** 1 (800) 722-3393  
[us.support@humanware.com](mailto:us.support@humanware.com)

**Europe:** (0044) 1933 415 800  
[eu.support@humanware.com](mailto:eu.support@humanware.com)

**Australia / Asia:** (02) 9686 2600  
[au.sales@humanware.com](mailto:au.sales@humanware.com)