
	Doc. Name: ACUITY REMOTE CONTROL CONCEPT	Doc. No.:	Rev.: 2.1
	Originator: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Original Date: 06/10/16
	Revision By: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Ref.: Skype Meeting

Remote to be used with Acuity Basic & Acuity Basic with Switch Screen

(a) Easy Controls:

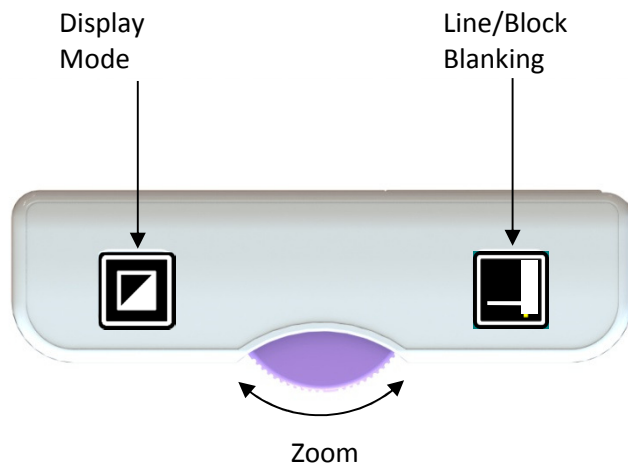

Display Mode

- Press to select the display mode.
- Press again to select the next display mode.
- Press and hold to toggle between high contrast and full colour mode and vice versa.


Zoom

- Rotate thumb wheel from left to right to zoom in.
- Rotate thumb wheel from right to left to zoom out.


Line/Block Blanking

- Press to select the required line or block blanking setting.
- Adjust the level of the required setting by rotating the thumb wheel.
- Press and hold the Line/Block Blanking button to exit this function and remove the line or block blanking off the screen.

NOTE: When the Line/Block Blanking is set then the thumb wheel defaults to work as zoom in or out.

	Doc. Name: ACUITY REMOTE CONTROL CONCEPT	Doc. No.:	Rev.: 2.1
	Originator: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Original Date: 06/10/16
	Revision By: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Ref.: Skype Meeting

(b) Advanced Controls:

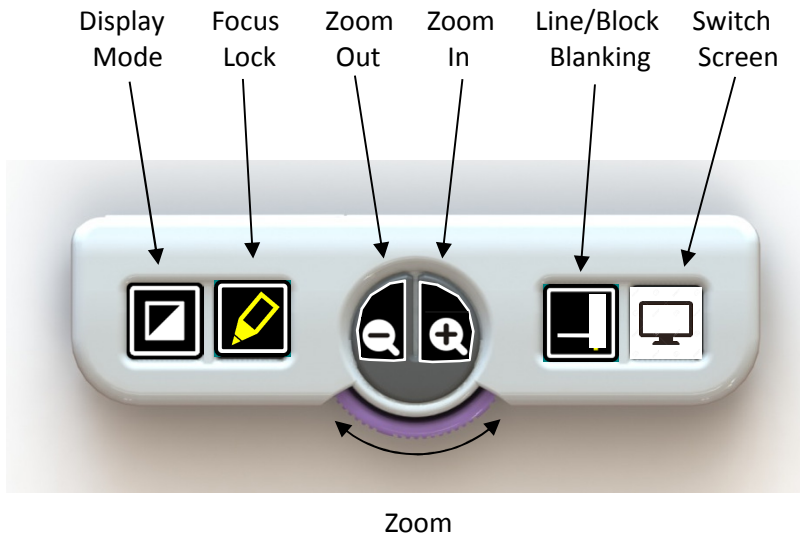

Display Mode

- Press to select the display mode.
- Press again to select the next display mode.
- Press and hold to toggle between high contrast and full colour mode and vice versa.

Focus Lock

- Press to enable focus lock.
- Press zoom in/out or rotate thumb wheel to disable focus lock.

Zoom

- Press left button to zoom out.
- Press right button to zoom in.
- Rotate thumb wheel from left to right to zoom in.
- Rotate thumb wheel from right to left to zoom out.


Line/Block Blanking

- Press to select the required line or block blanking setting.
- Adjust the level of the required setting by rotating the thumb wheel.
- Press and hold the Line/Block Blanking button to exit this function and remove the line or block blanking off the screen.

Switch Screen


- Press to activate switch screen when a computer is attached
- Press again to de-activate.
- ** For an Acuity Basic only this button will be blank.

NOTE: When the Line/Block Blanking is set then the thumb wheel defaults to work as zoom in or out.

	Doc. Name: ACUITY REMOTE CONTROL CONCEPT	Doc. No.:	Rev.: 2.1
	Originator: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Original Date: 06/10/16
	Revision By: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Ref.: Skype Meeting

Remote to be used with Acuity Basic with OCR

(a) Top Side:

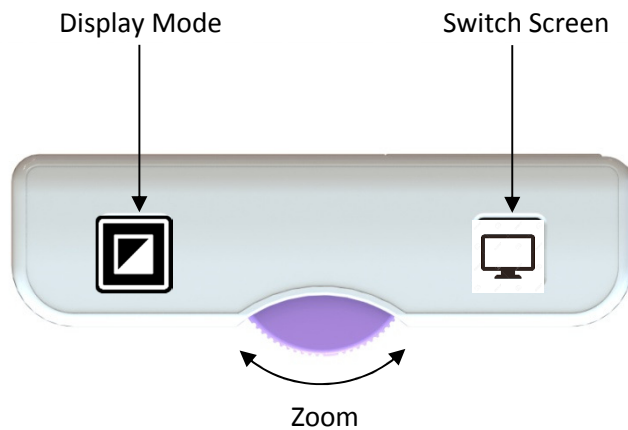

Display Mode


- Press to select the display mode.
- Press again to select the next display.
- Press and hold to toggle between high contrast and full colour mode and vice versa.

Zoom

- Rotate thumb wheel from left to right to zoom in.
- Rotate thumb wheel from right to left to zoom out.


Switch Screen








- Press to activate switch screen when a computer is attached
- Press again to de-activate.

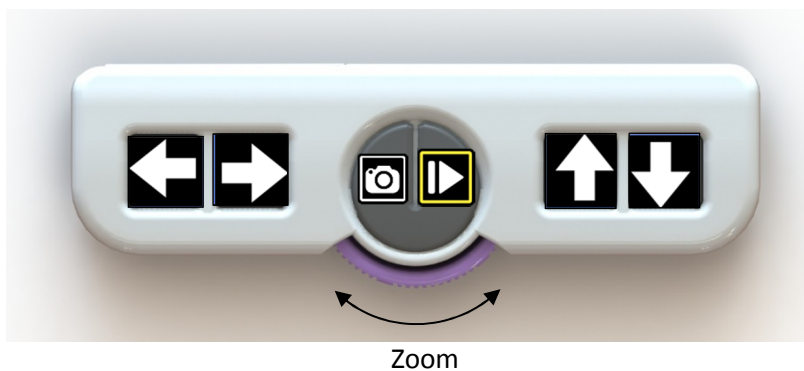	Doc. Name: ACUITY REMOTE CONTROL CONCEPT	Doc. No.:	Rev.: 2.1
	Originator: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Original Date: 06/10/16
	Revision By: RJ BLAKE	Function: QUALITY MANAGER, REHAN	Ref.: Skype Meeting

(b) Bottom Side:

Once you flip over the remote from the Top Side to the Bottom Side, then the Acuity toggles from live viewing mode to OCR mode.



	<ul style="list-style-type: none"> - Press to read previous word - Press and hold to read previous sentence
	<ul style="list-style-type: none"> - Press to read next word - Press and hold to read next sentence
	<ul style="list-style-type: none"> - Press to start a new scan - Press to stop reading in order to begin a new scan - Press to start the new scan
	<ul style="list-style-type: none"> - Press to pause reading - Press to re-start reading when reading is paused
	<ul style="list-style-type: none"> - Press to read the previous reading zone - Press and hold to navigate to beginning of present page and begin reading
	<ul style="list-style-type: none"> - Press to read the next reading zone - Press and hold to navigate to beginning of next page (if more than one page scanned). If there is no next page then the system navigates to end of present page and reading stops
	<ul style="list-style-type: none"> - Rotate the thumb wheel to zoom in/out