


No Vision Controls

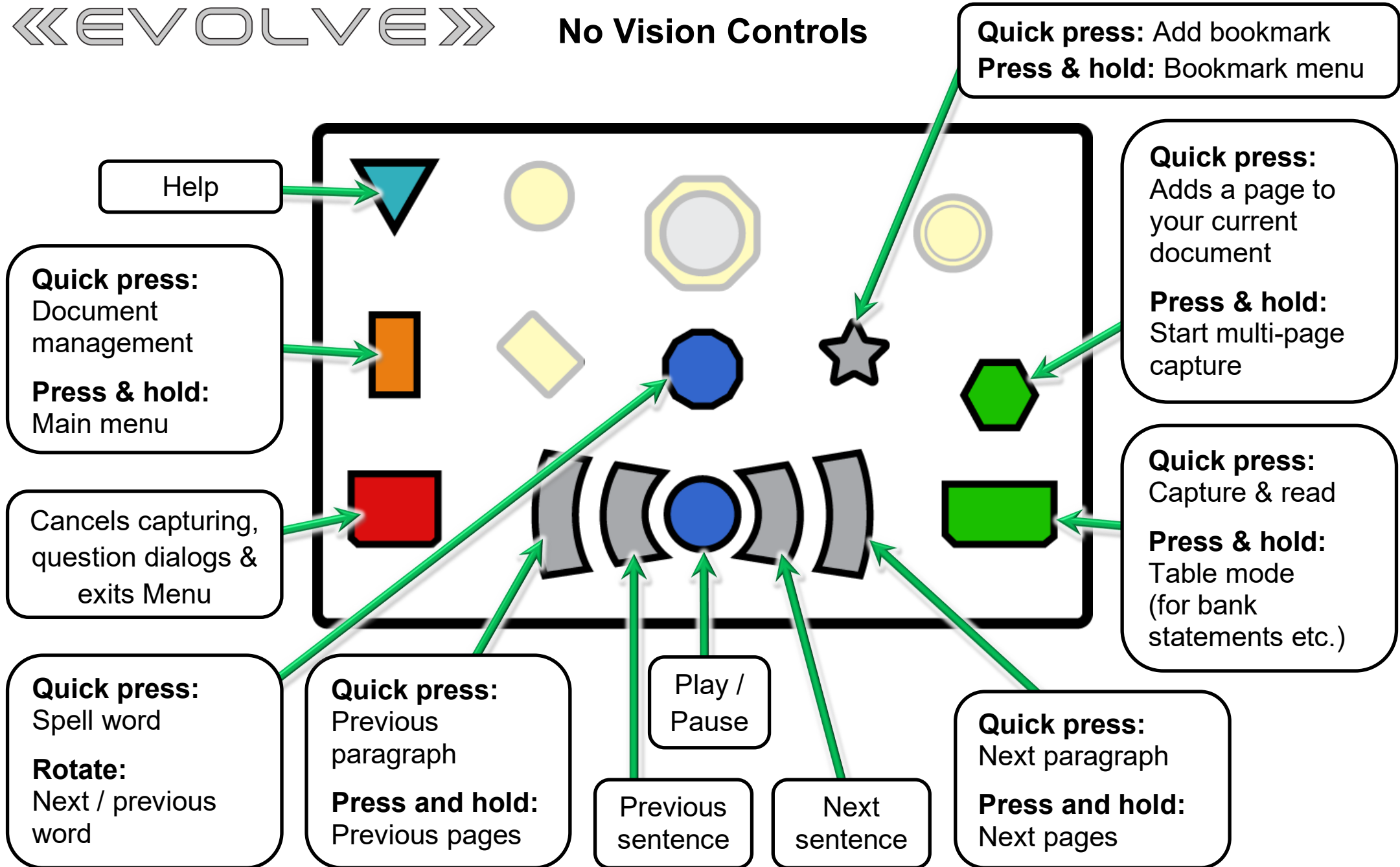

Low Vision Controls

In Document mode

Quick press:
Cycle text views

Press & hold:
See original image
(currently only in
Overlay view)

In Live mode

Rotate image

Quick press:
Switch between
Document mode
& Live mode


In Live mode

Press & hold:
Cycle between
Live mode &
Writing mode

Rotate:
Adjust
magnification

Quick press:
Cycle between preferred reading
colour, its inverse & full colour

**Press & hold & rotate
Magnification dial:**
Set preferred reading colours

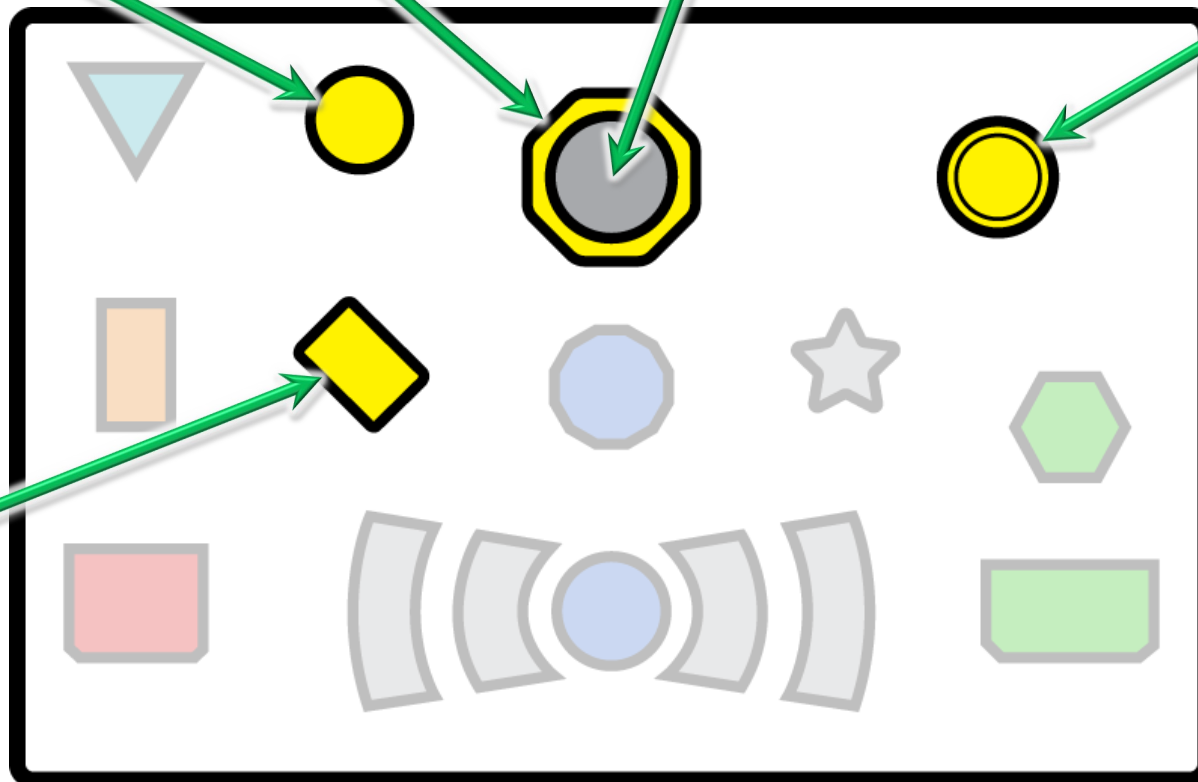

**In Live mode,
Image &
Overlay views**

Move:
Pan around
document

Quick press:
Start reading from
word nearest
pointer

**Reformatted
views**

Move:
Continuous
scrolling or read
screen-by-screen

Quick press:
Stop continuous
scrolling