


**VISUM**

**Ecare 3.5"**

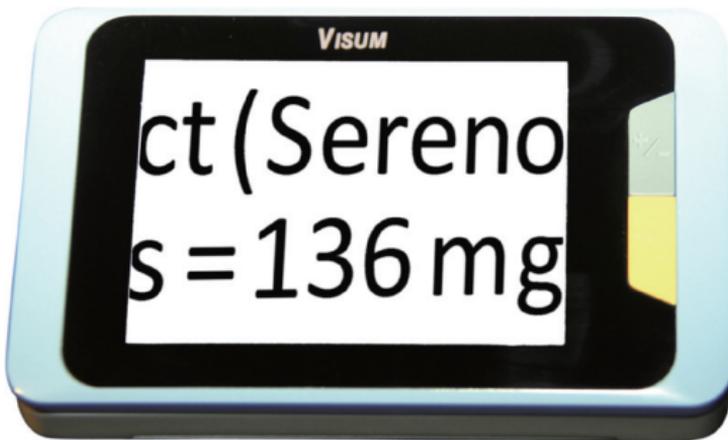

<http://www.daconevisum.com/>

# Index

Safety Precautions .....	1
Package Contents & Controls .....	2
Using the Ecare 3.5" .....	4
Power On/Off .....	4
Adjusting the Magnification .....	4
Selecting the Viewing Mode .....	5
Image Freeze Functions .....	5
Adjusting the Brightness Level.....	6
Turn LED Lighting On/Off .....	6
Turn On/Off the key tone .....	7
Memory Function .....	7
TV Out .....	7
Battery Indicator .....	8
Writing Function .....	8
Specifications .....	9
Warranty .....	10

Congratulations on your purchase of the Ecare 3.5" from Beijing Outsmarting Dacone. The Ecare 3.5" is a handheld electronic video magnifier that you can take anywhere.

The Ecare 3.5" allows you to read labels, maps, menus, and perform a wide variety of visual and writing tasks with ease.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or Beijing Outsmarting Dacone's headquarters.

We hope you will enjoy the convenience of the Ecare 3.5".

## Safety Precautions

- Do not expose the Ecare 3.5" to extreme or rapid changes in temperature or humidity.
- Do not use the unit under  $-5^{\circ}\text{C}$  or above  $+40^{\circ}\text{C}$ .
- Do not place Ecare 3.5" near a radiator, and do not expose it to direct sunlight, as it may damage electronic and optical components of the Ecare 3.5".
- Please handle the device with care. Do not shake, squeeze, or drop it, as it may damage internal components.
- Do not use cleaning agents or abrasive materials to clean the Ecare 3.5", as it may damage the product surface and internal components. Avoid touching the lens of the camera, and only clean it using specialized cleaning materials.
- Do not unscrew or open the device, as this will instantly void its warranty. Under warranty conditions, only certified Dacone repair specialists can service the Ecare 3.5". Please contact your local Dacone distributor if you have any questions.
- Only use the power adapter provided with the Ecare 3.5". Using any other adapter may damage the device and voids its warranty.
- Use of the Ecare 3.5" for means other than those described in this user manual will void its warranty conditions.

## Package Contents&Controls



Please refer to the illustrations below to identify the package contents and controls of your Ecare 3.5" device. In addition to this user guide, your Ecare 3.5" package should contain the following items:

- Ecare 3.5" (Battery included inside the device)
- Wrist Strap
- Carrying Case
- Lens Cloth
- Video Cable
- A/C Adapter
- Reading Stand
- User Manual

Note: if any of the items above are missing from your Ecare 3.5" package, please contact your Dacone distributor immediately.

 Ecacare 3.5"	 Wrist Strap	 Carrying Case
 Lens Cloth	 Video Cable	 A/C Adapter
 Reading Stand	 User Manual	

Please note the locations of the following controls on your Ecare 3.5", as displayed below:

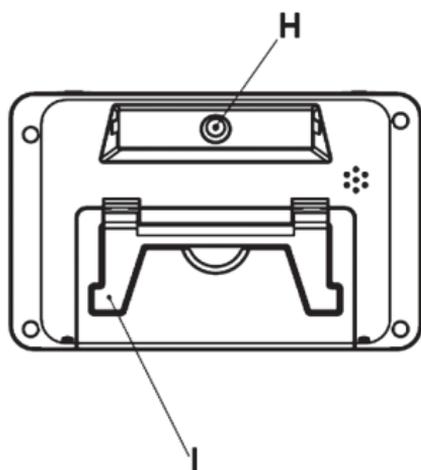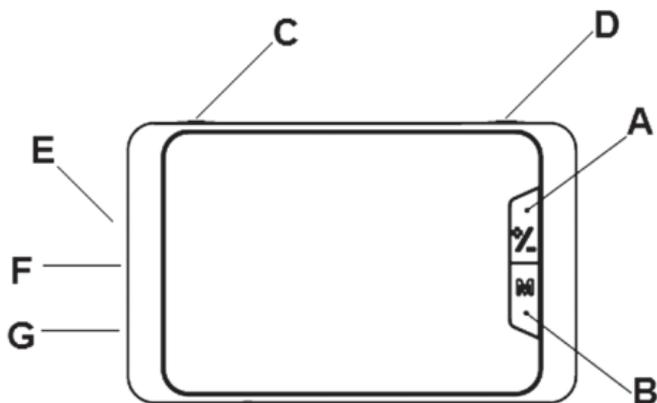

- A. Zoom +/-
- B. Mode Button
- C. Power Button
- D. Freeze Function
- E. Battery Indicator
- F. DC In
- G. TV OUT
- H. Camera Lens
- I. Reading Stand


## Using the Ecare 3.5"

### Power On/Off

Power on: to turn on the device, press and hold the Power Button for 3 seconds.

Power off: to turn off the device, press and hold the Power Button for 3 seconds.

When inactive for 3 consecutive minutes, the device will be automatically turned off.

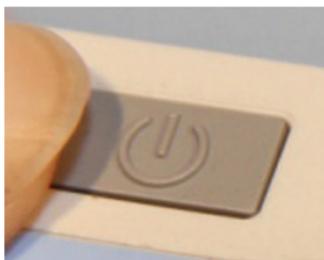

### Adjusting the Magnification

Press the Zoom Buttons (labeled "-", "+") to adjust the digital magnification level (4 different levels).

The total combined optical and digital magnification ranges from 2.5x to 17x. To adjust the magnification to particular size, users may also adjust the distance between the camera lens and the reading material.

When using the Reading Stand, the magnification level ranges are 3.6x, 7x, 11x, and 14x.



## Selecting the Display Mode

To select the preferred display mode, please press the Mode Button (Green) to alternate between different display modes.

Ecare 3.5" has 8 Display Modes available:

- Full Color Photo Mode
- Grey Color Mode
- White on Black Mode
- Black on White Mode
- Blue on White Mode
- Yellow on Black Mode
- Yellow on Blue Mode
- Blue on Yellow Mode



## Image Freeze Functions


Press this "Freeze" button to freeze an image on the display. You can also adjust the magnification and color modes after freezing an image.

Press the button again to exit freeze image mode.



## Adjusting the Brightness Level

To adjust the brightness level, press and hold the Freeze Button for 3 seconds to access the brightness menu, and use the "+/-" Button to adjust the brightness level. Once the desired brightness level has

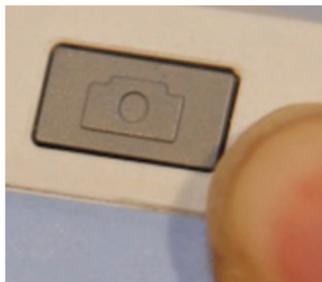

been selected, press and hold the Freeze Button once again to confirm.

Note: all other buttons do not function when the Brightness Menu is enabled.

## Turn LED Lighting On/Off

By default, the LED Reading Light is enabled as the device is turned on.


To turn on/off the LED Reading Light, press and hold the Mode Button for 3 seconds.



## Turn On/Off the Key Tone

By default, the Key Tone is enabled as the device is turned on.

To turn on/off the Key Tone, press and hold the Mode Button for 3 seconds.

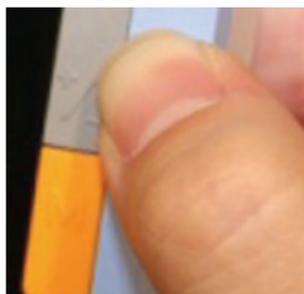

## Memory Function

The Ecare 3.5" Memory Function enables all display parameters (magnification level, display mode, brightness level, and key ton function) to be saved as the device is powered off. These parameters will automatically be applied once the device is turned back on.

## TV Out

The Ecare 3.5" TV OUT Function allows connecting the device to a TV display, via TV cable, so as to display the device display onto a TV screen.

## Battery Indicator

A battery indicator is located on the upper right corner of the device. Its described features are as follows:

- Flashing Red Light: battery needs to be recharged
- Non-Flashing Red Light: battery is being re-charged
- No Light: battery is full recharged

## Writing Function

Using the Ecare 3.5"’s Reading Stand, the device can be used for writing. As shown in the picture below, the device can be used to conveniently write sentences and sign documents.



## Specifications

Magnification Range	2.5X-17X
LCD Screen	3.5" TFT LCD
Magnification Level	4
Display Modes	8
Image Freeze Function	Yes
Adjustable Brightness	Yes
LED Light On/Off	Yes
Key Tone On/Off	Yes
TV Out	Yes
Folding Stand	Yes
Continuous Working time	4 hours
Weight	140g
Dimensions	118 x 74 x 19 mm

## Warranty

Beijing Outsmarting Dacone offers a 2-year Limited Warranty from the date of original purchase. Always keep your purchase receipt safely, as it may be needed for warranty claim purposes.

- Assuming the problem was caused by defects under normal and prescribed device use, Dacone will either repair or replace the device (at company's option), without charge.

- This warranty is not applicable in the following circumstances:

- Intentional damage, or damage caused by excessive physical force.

- Using device under extreme conditions (e.g. excessive heat) or against device instructions (e.g. using an incorrect power adaptor). For more information on terms and conditions, please refer to the Safety precaution chapter of this user manual.

- Scratches and blemishes from normal use.

For any further questions on our Warranty Policy, Terms, and Conditions, please contact your local Dacone distributor with dated proof of purchase, or send us an email at [service@daconevisum.com](mailto:service@daconevisum.com). Your claim will be answered within 48 hours.

