

OrCam MyPro - Smart Reading Commands

Smart Reading

This command starts the OrCam MyPro smart reading feature.

Help

This command will have the OrCam MyPro tell you all the available commands.

Read everything

This command will have the OrCam MyPro read everything within its field of view.

Start from "X" / Jump to "X"

Both of these commands will have the OrCam MyPro find the first instance of the word you spoke and start reading from the beginning of the sentence.

Read the dates

This command will have the OrCam MyPro read every date found in the text.

Read the phone numbers

This command will have the OrCam MyPro read every phone number found in the text.

Read the amounts

This command will have the OrCam MyPro read every currency amount found in the text. The current supported currencies are:

\$ Dollar currencies (including USD)

£ Euro

€ British Pound

¥ Japanese yen

Previous

This command will have the OrCam MyPro read the previous instance.

Next

This command will have the OrCam MyPro read the next instance.

Repeat

This command will have the OrCam MyPro repeat the last sentence.

Read the headlines

This command will have the OrCam MyPro find all headlines in the text and read them one after the other.

Read article number X (X can be numbers 1 to 10)

After reading the headline, speaking this command will have the OrCam MyPro read whichever article number you requested.

Read the first article/second article (up to tenth article)

Same as "Read article number X".

Read the article about "X"

This command will have the OrCam MyPro find and read the article that contains the word you say.

Note: "X" is a placeholder for the word you are searching.

Restart

Should the Smart Reading feature not be functioning properly, you can say this command to restart the mode instead of exiting and re-entering.

Volume Up

OrCam MyPro will raise the device's volume.

Volume Down

OrCam MyPro will lower the device's volume.

Reading Speed Up

OrCam MyPro will now speak and read faster.

Reading Speed Down

OrCam MyPro will now speak and read slower.

Exit

This command will exit the OrCam MyPro's Smart Reading feature.

If you have any questions or require any further training or support please call VisionAid on 01775 711 977 or email hello@visionaid.co.uk