

# HELIX HD

## Quick Reference Guide – Control Pad

**Quick Press:** Switch between the Helix HD or another video device (if connected)

**Quick Press:** Displays last used guide / Cycle through different reading guides  
**Rotate:** Resize reading guides  
**Press & Hold:** Turns off reading guides

**Rotate:** Adjust zoom  
**Quick Press:** Handwriting mode  
**Press & Hold:** Overview mode

**Quick Press:** Change between full colour & enhanced reading colours  
**Rotate:** Cycle reading colours

**Quick Press:** Enter / exit Settings. Once in Settings use Colour Mode dial to select settings

**Rotate:** Adjust contrast  
**Quick Press:** Turn lights on / off  
**Press & Hold:** Freeze image



# Simplified Control Pad



## Magnetic overlay:

Place over the 4 controls as shown to further simplify

## Replacing control pad batteries:

Turn the control pad over & remove the battery cover by pushing the centre ridges and sliding them away from the unit. Insert two AAA batteries as shown on the slide out battery cover guide (+ve to the right)



Your Distributor:

**VISION**Aid

**VisionAid Technologies Ltd**  
Bridge Lodge, Spalding Common,  
Spalding, Lincolnshire, PE11 3AU

**Tel:** 01775 711 977  
**e-mail:** [info@visionaid.co.uk](mailto:info@visionaid.co.uk)  
**Web:** [www.visionaid.co.uk](http://www.visionaid.co.uk)