

Zoomax Snow - Getting Started Guide

Battery

Before using your Snow for the first time, please ensure the battery is fully charged. To do this, insert the tip of the charger into the charging socket on the bottom side of the Snow (as shown overleaf).

Once charged, the red charging light on the front of the Snow will no longer be illuminated.

To remove the battery, (to replace it or for a hard reset of the Snow), slide the cover off as detailed below to give access to the battery compartment.



Handwriting stand


To extend the Snow's integrated handwriting stand, simply place your index finger under the finger cut-out in the centre of the handwriting stand and pull it upwards.



Snow Controls

On / off (press and hold for 2 seconds)

TOP VIEW

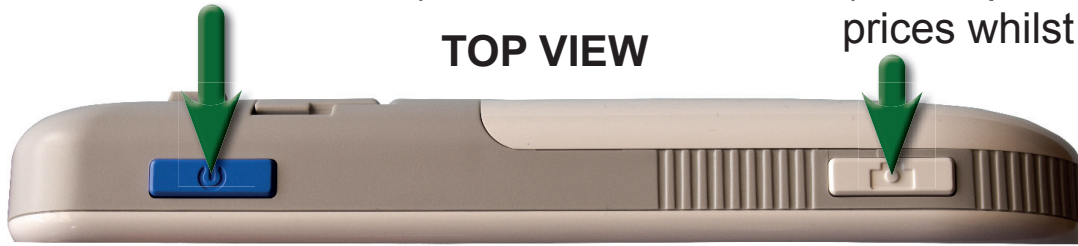

Freeze frame
(for telephone numbers / prices whilst shopping)

FRONT VIEW



Charging LED
Solid Red: Charging
Out: Fully charged

Zoom in and out
Tap for steps or press & hold for continuous

Colour change
Short press: Cycle colours in current mode.
Long press: Toggle "standard" and "favourites mode".



BOTTOM VIEW

Charging socket

TV connection socket
(used to connect Snow to phono / RCA socket on TV)

Wrist strap hole

For more information, please contact

VisionAid

VisionAid Technologies Ltd

Bridge Lodge, Spalding Common,
Spalding, Lincolnshire. PE11 3AU.

Tel: 01775 711 977

Web: www.visionaid.co.uk

Email: sales@visionaid.co.uk