

**Anyview® IView5**  
**Handheld Video Magnifier**

**User Guide**



Anyview for Any Place.






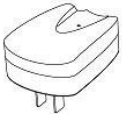

## Content

1. Parts and accessories.....	4
2. Structure.....	<b>Error! Bookmark not defined.</b>
3. Using Guide.....	5
3.1 Correct use of magnification.....	<b>Error! Bookmark not defined.</b>
3.2 Correct use of writing stand.....	<b>Error! Bookmark not defined.</b>
3.3 Functions Operation.....	6
4. Troubleshooting.....	9
5. Safety Guide.....	<b>Error! Bookmark not defined.</b>
6. Technical Specifications .....	11




## **Description**


Anyview® IView5 inch handheld video magnifier is of integrated design for writing stand and reading handle, which makes the writing and reading convenient. With excellent functions and special color technology, this magnifier brings you outstanding vision enjoyment. Its humanized power on/off design makes the magnification easy without pressing the button.
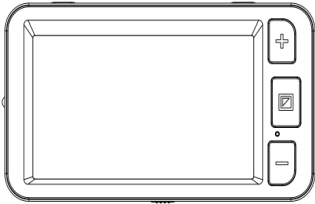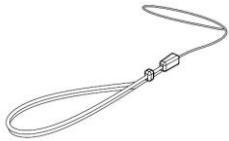
## 1. Parts & Accessories

 <p>Video Magnifier</p>	 <p>Short Strap</p>	 <p>User Manual</p>
 <p>Video cable</p>	 <p>Protective case</p>	 <p>AC power adapter</p>
 <p>USB cable</p>		

## 2. Structure

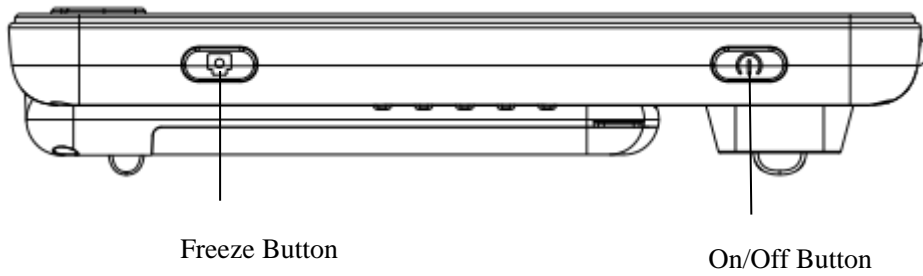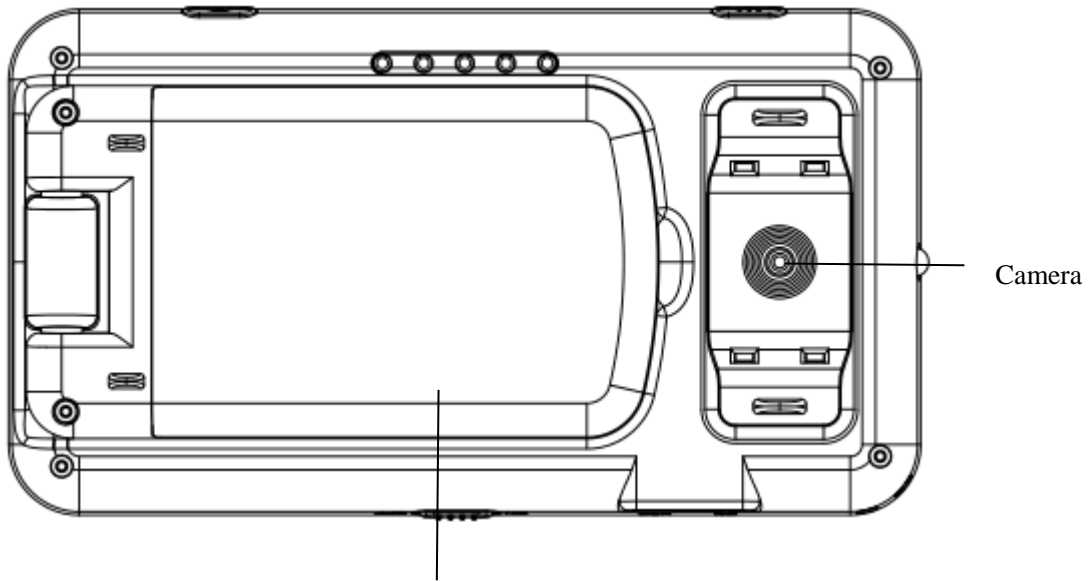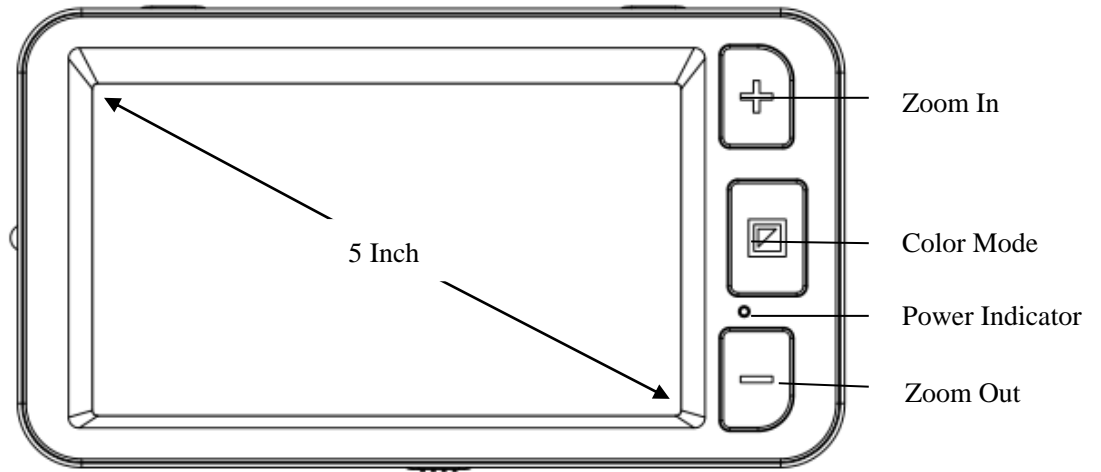

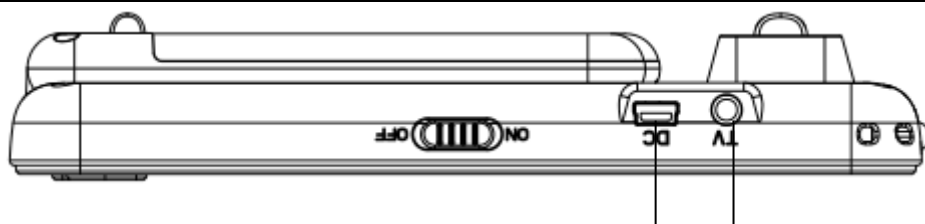

Power Button TV output port

### 3. Using Guide

#### 3.1 Functions operation


##### 3.2.1 Power On/Off function



Power On/Off Button


- Press and hold the power button for 3 seconds until the BEEP heart to turn on, the screen will show the opening image for 2 seconds and then the unit will enter into video magnifier mode
- Press and hold the power button for 2 seconds to turn off.

##### 3.2.2 Zoom In/Zoom Out

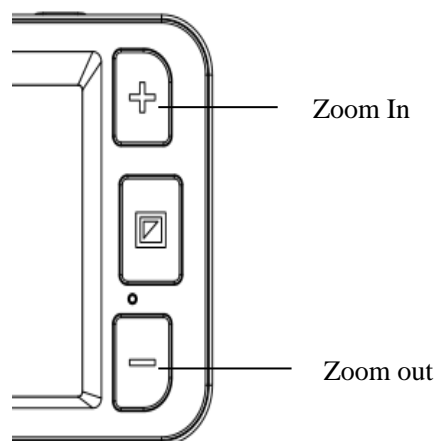

- Press the “+” button to zoom in, and press the “-”button to zoom out , you can choose the right and preferred magnification.

### 3.2.3 Contrast color mode and Camera lighting function

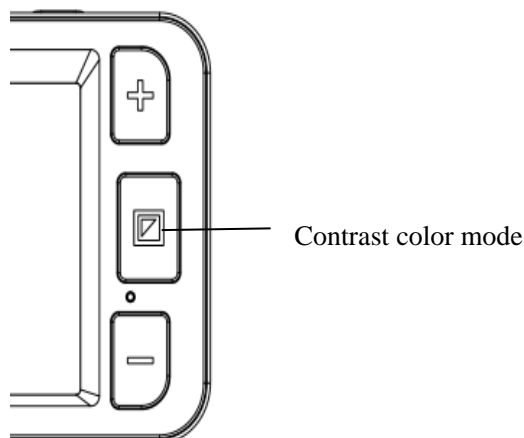

- Normal mode and favorite mode are available for contrast color mode.
- Press and hold the contrast mode button for 2 seconds can switch between the normal mode and favorite mode.
- In favorite mode, press the mode button to cycle through 3 basic modes: full color, your favorite contrast color and the reverse (such as white on blue & blue on white)
- In Normal mode, press the mode button to cycle through the following 10 color modes:
  - Full color mode
  - Grey mode
  - High contrast white text on black background
  - High contrast black text on white background
  - High contrast white text on blue background


- High contrast blue text on white background
- High contrast yellow text on black background
- High contrast black text on yellow background
- High contrast blue text on yellow background
- High contrast yellow text on blue background

● Press and hold the contrast color mode for 5 seconds to turn on/off the camera lights.

### 3.2.4 Brightness Adjustment.

- Hold the “Freeze” button and “Zoom in” button together to enter into brightness adjustment.
- A brightness bar shows, then press “Zoom in” to increase brightness and “Zoom out” to decrease it.

### 3.2.6 Freeze Function

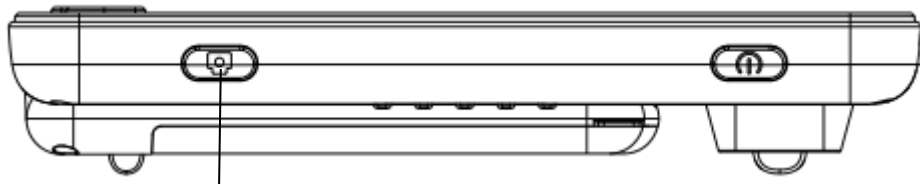

Freeze Function

- Use Freeze button to capture images and letters, Take the unit closer to have a carefully look at the object, and you can also magnify or minify images by +/- button and switch contrast modes by color Mode button
- Press the Freeze button again to release

### 3.2.7 Setting memory function

- The unit will memorize automatically the most recent settings



(e.g. color mode, magnification, video output format, button beeps, brightness) and use them in the next power on.

### 3.2.8 Power saving function

- Under magnification mode, the unit will turn off automatically if no operation for over 3 minutes.

### 3.2.9 TV output function

- If the unit screen size is not enough for you, then you can use the TV output function to connect to the TV screen for magnification.
- After the TV output connected to TV, turn on the unit, the image will automatically switch on the TV screen.

## 4. Troubleshooting

Troubles	Solutions
Black screen	<ul style="list-style-type: none"> <li>● to check if it is on or off</li> <li>● To check if the camera is covered by dirty spot.</li> <li>● Decrease the magnification</li> </ul>
Unable to turn on	<ul style="list-style-type: none"> <li>● To check if the battery is in or if the battery has the power.</li> <li>● Try to use the power adapter to turn on.</li> <li>● Press the power button</li> </ul>

	<p>together with color mode button for 3 seconds to turn on</p>
<p>Image darkness and twist on the screen</p>	<ul style="list-style-type: none"> <li>● To check if the reading material is laid flat.</li> <li>● To check if the camera lights are off.</li> </ul>
<p>No display after connected to TV</p>	<ul style="list-style-type: none"> <li>● To check if the TV is on.</li> <li>● To check if the wire is correctly TV output and connected to TV.</li> <li>● To check if it is the correct TV output mode.</li> </ul>
<p>Unable to recharge</p>	<ul style="list-style-type: none"> <li>● To check if the power adapter is connected well.</li> </ul>

## 5. Safety Guide

To keep the video magnifier in good conditions, please read the following safety guide carefully:

- Do not expose IView5 to excessive heat or direct sunlight to avoid the risk of fire.
- Do keep away from the humidity, rain, liquid and chemicals.
- Do not use this magnifier near inadequately shielded medical devices.

- Do use this magnifier under the temperature between 10° C and 40°C
- Do not try to repair the magnifier by yourself or open the its shell as it will void the warranty. Contact your dealer to service this equipment if necessary
- Always unplug the system before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.
- Do use its matched adapter to recharge, otherwise it will damage the unit.

## **6. Technical Specifications**

- 5-inch widescreen LCD
- 2.2x -25x continuous zoom
- Focus: Auto focus
- TV Out: PAL/NTSC
- Power saving mode: the unit will turn off automatically after inactivity for 3 minutes
- Easy flip-out handle with writing function
- Fast reading with no blur
- Charging time: About 5 hours

- Functions: Freeze image, Switch color mode, Zoom in/out
- Settings saved after power-off
- Lighting function
- Battery level indicator : Positive
- Lightweight design at 230g.
- Battery: rechargeable lithium battery 4 hours continuous use
- Accessories: user manual, AC power adapter, short strap, protective case, video output cable
- Operation and storage conditions: Humidity<70% Temperature: 10-40°C
- 10 high-contrast colour viewing modes (3 favorite color modes)

## **Warranty**

- Main unit body one year
- Power adapter and accessories one year.
- Coming upon the troubles, please contact with after-sale service. If impossible to solve it by yourself, please read carefully the warranty card, and post the unit together with the card to us, feedback will immediately forwarded once the item received.



