



Empowering Your Vision



User's Manual

IMPORTANT SAFETY INSTRUCTIONS

READ THESE INSTRUCTIONS

CLEAN WITH A DRY SOFT CLOTH

***DO NOT WALK OR DRIVE WITH THE
E2+ BY NUEYES***

**DO NOT EXPOSE INTERIOR
LENSES TO SUNLIGHT**

***CHARGE YOUR BATTERY
BEFORE USE***

What's Included

**CAREFULLY UNPACK AND
CONFIRM THE FOLLOWING
PARTS ARE INCLUDED**

1. Your e2+ Device
2. Cleaning Cloth
3. Block Charger
4. Charging Cord
5. Remote Control

How to Put On Your e2+ For Maximum Clarity And Comfort

1. Please turn on your e2+ by pressing the power button on the below right-hand side of the product



****PLEASE NOTE OUR APPLICATION
WILL TAKE 10-12 SECONDS TO
LOAD. DO NOT PRESS ANY
BUTTONS****

2. Align the lenses/screen to your eyes
3. Once aligned gently place the strap over your head
4. Once strap is on your head, adjust the side straps tighter or looser for maximum comfort and clarity

*****PLEASE NOTE IF THE IMAGE DOES NOT LOOK CLEAR TO YOU ADJUST THE e2+ UP OR DOWN ON YOUR FACE*****


Magnifying with Your e2+

1. To increase magnification, press the top button on the right-hand side of your e2+



When you reach the maximum magnification, an icon displaying MAX will appear.

2. To decrease magnification, press the middle button on the right-hand side of your e2+

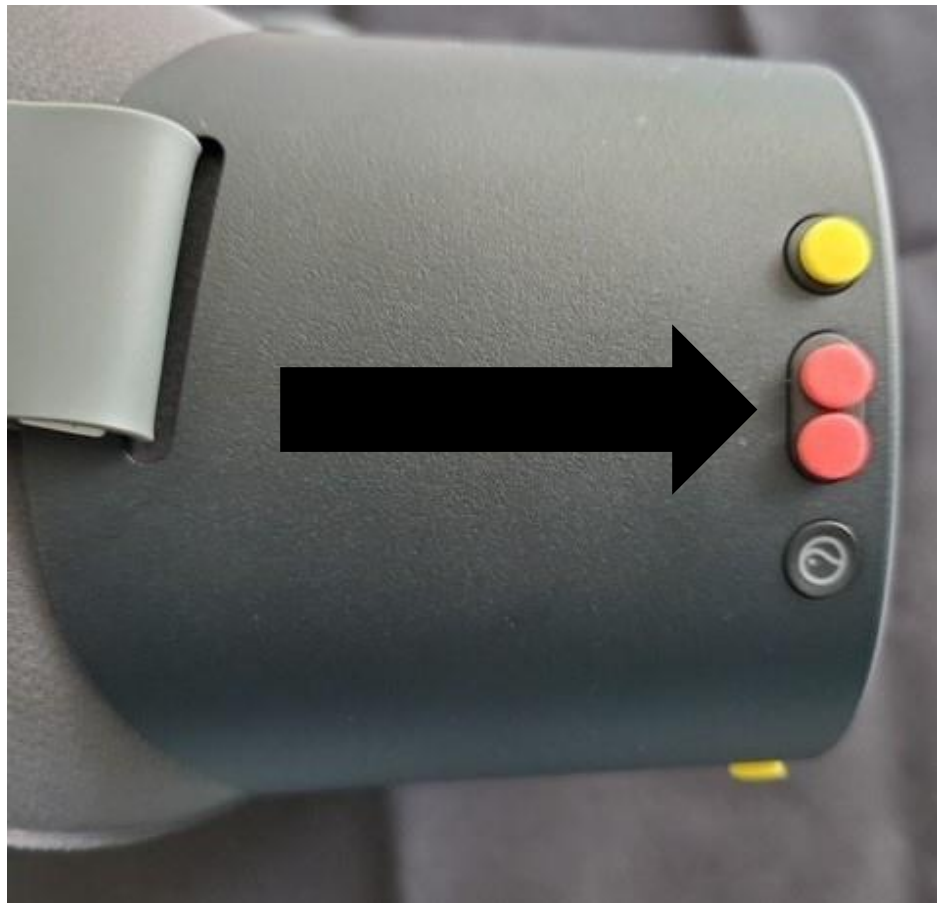

Once at lowest magnification an icon displaying the word **MIN** will appear

Changing Contrast with Your e2+

1. To change contrast: Press the Left or Right underside (Volume) button of your e2+. The contrasts will cycle through **5 levels of contrast.**



Using OCR-Text To Speech with e2+

1. To use the OCR feature: Press and hold down the magnification button for 2 seconds. The OCR will start on its own



E2+ Menu

1. To access your e2+ Menu: Press and hold down the contrast/volume or left underside button for 2 seconds



2. Once in the menu you have four options.
 - **Vision** (Brightness control 40-100, Language)
 - **OCR** (Gender voice, Speed, Volume, Text Color, Highlight Color)
 - **Settings** (Where to set up WIFI)
 - **Update** (OTA - Over the Air)
3. To select each option, press the middle right button (decrease magnification)
4. To toggle an option, press the top right button (increase magnification button)

5. To exit please hold the contrast button for 2 seconds.

Turning OFF/ Powering Down Your e2+

1. Simply press and hold the power button for 2 seconds. The NuEyes logo will appear (Chimes) Screen will turn black indicating the device is off.



2. Take device off or Lay down the device to go into “sleep mode”

Charging Your e2+

Simply plug your USB cord into the slot next to the power button.



1. A light appears next to the charging plug indicating it is charging. (Red Light needs charge) if Red Light does not change or device does not charge (Hold power button for 10 secs, this will reboot the device)



2. Blue light, charging - Forest green light, fully charged. (30min to 1 hour)

3. If you are not sure it is charging, please (quick press power button) look in screen, large white battery shows charging.

Rebooting Your e2+

Press power button for 10 seconds

Turn device on by pressing power button

NuEyes app will load

**Please find e2 product
line training video**

**[https://youtu.be/-
79ImvpUmeU](https://youtu.be/-79ImvpUmeU)**

www.nueyestech.de

e2 product tab

Quick Reference Guide:



- Yellow Dot Bottom right **POWER BUTTON**
- Orange Dot Bottom mid **MARKER FOR USB PLUG IN**
- Red Dots Bottom left **e2+ HOLD 2 SEC QUICK MENU/ QUICK PRESS CONTRAST**



- Upper Right-Hand Yellow Dot **INCREASE MAGNIFICATION/TOGGLE** feature for Quick menu
- Upper Right Hand 2 Red dots (1 button) **DECREASE MAGNIFICATION/SELECT** feature for Quick menu

Technical Specifications

Display	4K
CPU	Qualcomm Snapdragon 835
Weight	298g (w/o Band)
Refresh rate	75hz
Storage	128G
Battery	3-5 Hours (depending on heavy usage)
Tracking	3DOF Camera
Camera	16mp