

# VisionAid - HandyReader HD v2

## Quick Reference Guide

---

### Top



**On / Off:**  
Press &  
hold for 2  
secs

**Freeze:** Quick press  
**Reading Line On / Off:** Press & hold 2 secs  
to cycle between horizontal, vertical & off  
**Select / Deselect:** Quick press when in Menu

---

### Front



**Zoom In / Out:**  
Press & hold  
**Adjust Reading  
Line:** Press & hold  
whilst line flashes  
on screen


**Change Colour  
(M):** Quick press

**Left LED**  
Green:  
On

**Right LED**  
Red: Charging  
Off: Charged

**Settings Menu:** Press & hold for  
2 seconds to open & close  
**In Menu:** Quick press to cycle  
through options

## Back

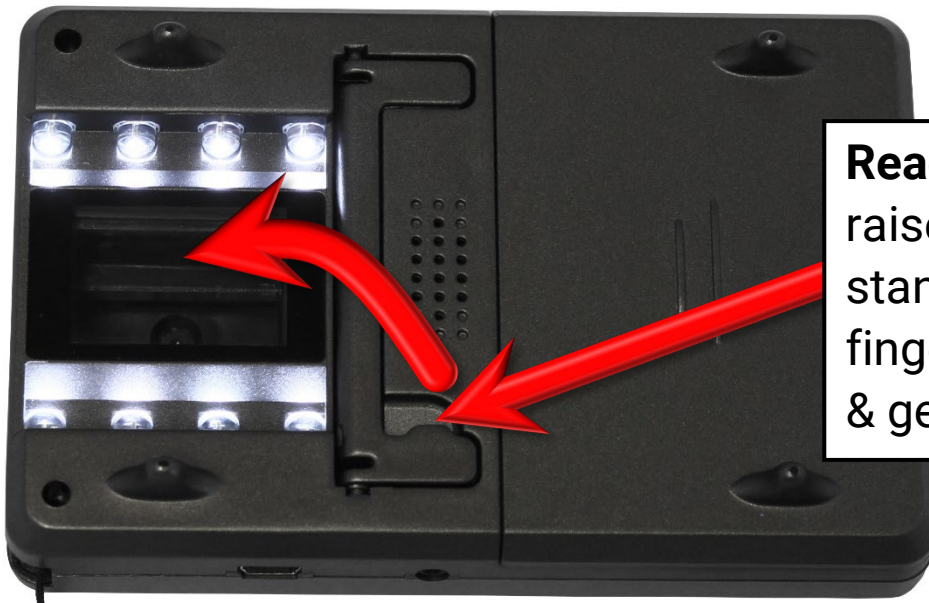

**Reading stand:** To raise the reading stand, insert your finger into the cut out & gently lift it upwards

## Bottom



**Lanyard bar**

**TV Out:** Connect the included TV out cable to this socket & your television to magnify onto your television screen  
Ensure your HandyReader HD & television are switched on. Then ensure you set your television to the correct source (normally A/V)

**DC In:** To recharge your unit, insert the plug gently into this socket

# VisionAid International

**VisionAid International Ltd**  
Bridge Lodge, Spalding Common,  
Spalding, Lincolnshire, PE11 3AU

**Tel:** +44(0)1775 711 977  
**Email:** [info@visionaid.com](mailto:info@visionaid.com)  
**Web:** [www.visionaid.com](http://www.visionaid.com)