



READEASY «EVOLVE»

Quick Reference Guide v1.0

Connecting the Camera

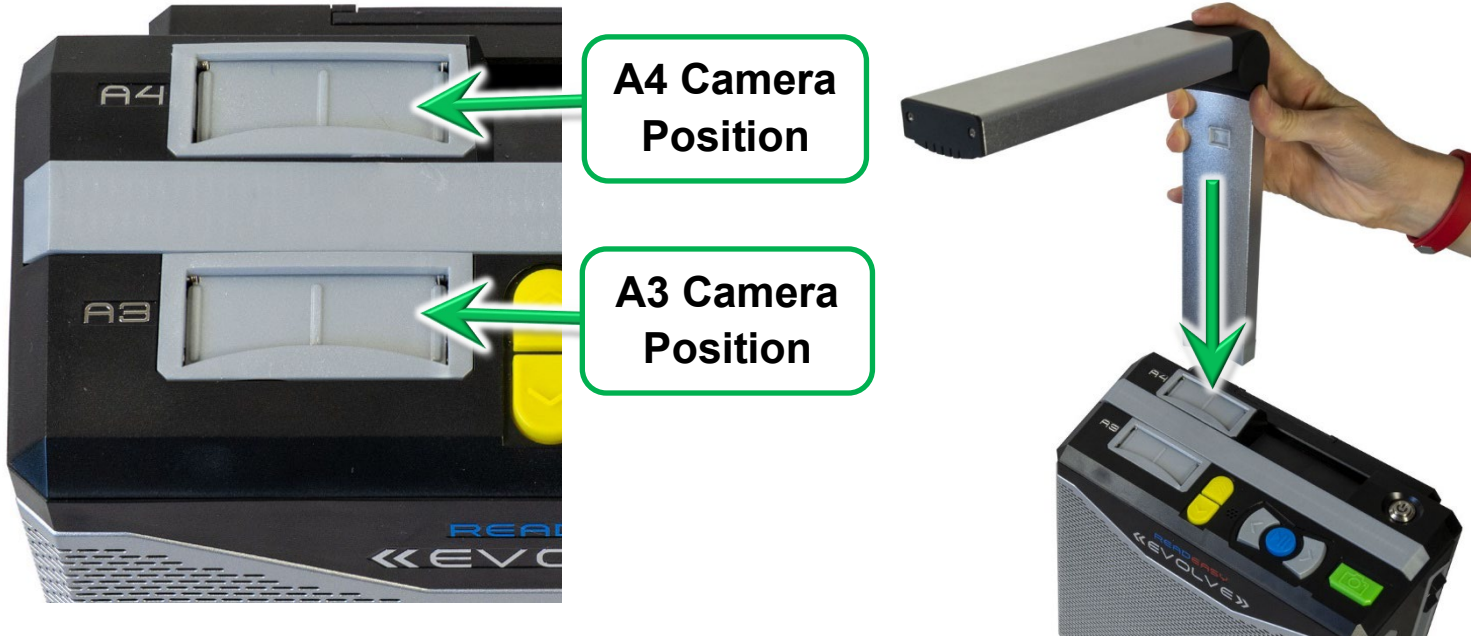

Standard Controls

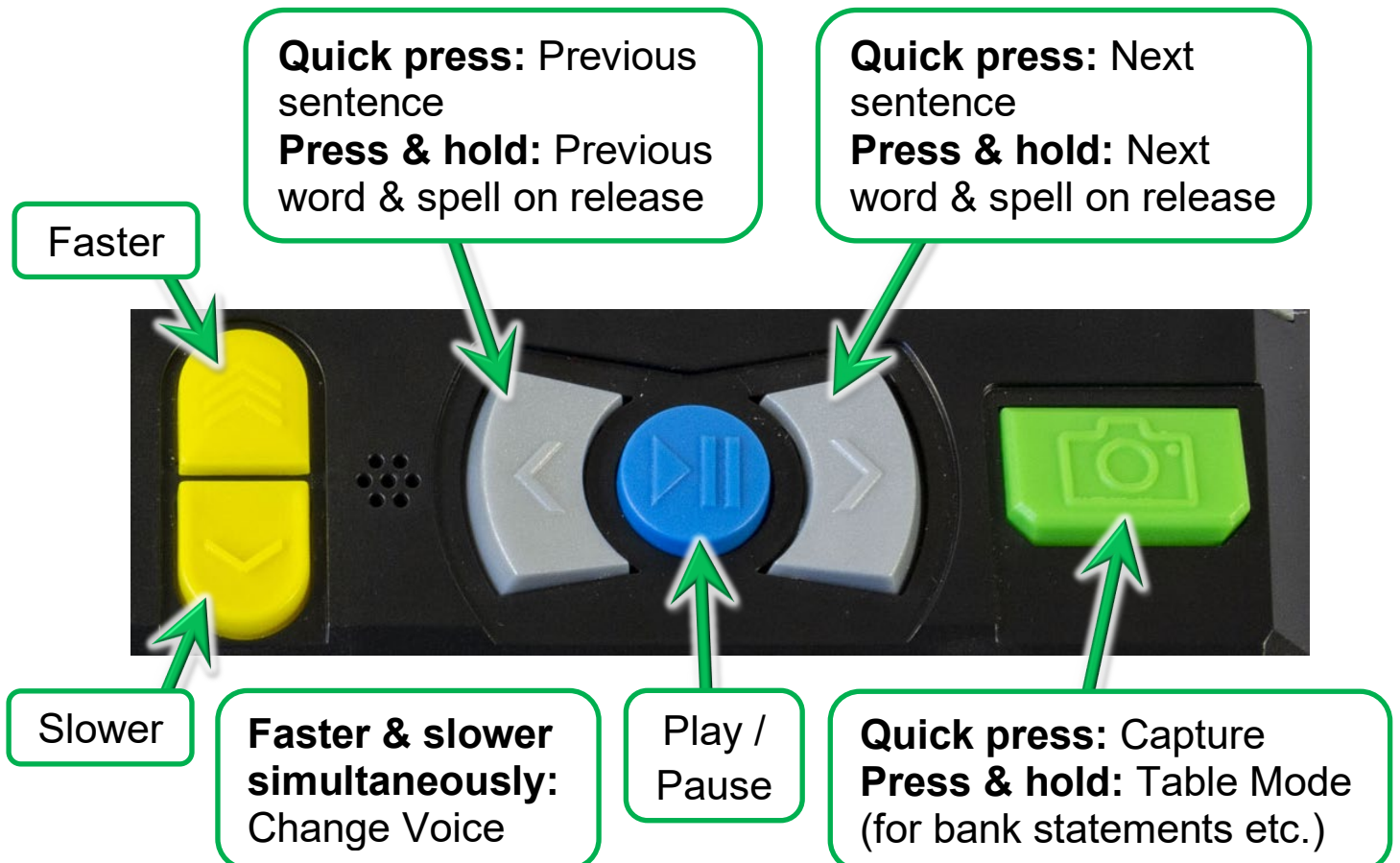

Additional Controls

For additional controls, such as enhanced navigation and changing recognition profiles, please see page 19 onwards of the user manual.

On / Off, Volume and Headphones

On / Off

To turn the ReadEasy Evolve on, press the on / off button, located on the top back right of the machine. It is a round button which is recessed in a round hole.



Volume Slider

To increase or decrease the volume, simply raise or lower the volume slider. This is located on the right-hand side of the machine.



**Headphone Socket
(3.5mm Stereo)**

To use the included headphones (or any other that you may already own), simply plug them into the headphone socket, located on the right-hand edge of the machine.

Contact Information

If you have any queries with the operation of your ReadEasy Evolve, please contact your local distributor.

Designed and Manufactured by

VisionAid International Ltd

Bridge Lodge,
Spalding Common,
Spalding, Lincolnshire
PE11 3AU
United Kingdom

Telephone:

+44 (0)1775 711 977

Email:

sales@visionaid.com

