



Hillrom™

BRAUN

ThermoScan®
PRO 6000



80028341 Ver. A
Revision Date: 2021-04
©2021 Welch Allyn, Inc.
All rights reserved.



This product is manufactured by Welch Allyn, Inc. under licence to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany. ThermoScan is a registered trademark owned by Helen of Troy Limited.


Hillrom Technical Support

hillrom.com/en-us/about-us/locations/

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

Cleaning guide

Thermometer body

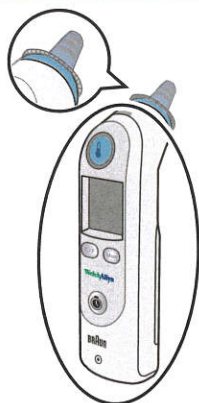


NOTE Place a new probe cover on the thermometer before cleaning.

Approved cleaning solutions

- 70% isopropyl alcohol or ethyl alcohol
- Hydrogen peroxide (Virox, Oxivir)
- Quaternary ammonium compounds (CaviWipes™, Clinell Universal Wipes®, Sanicloth)
- 10% Chlorine bleach solution


1

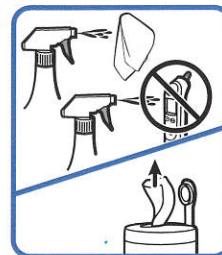

2




3

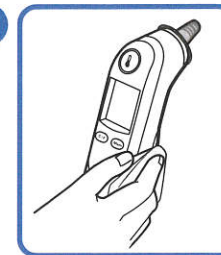

Wipe dry. Allow 5 minutes drying time.
Make sure thermometer body is dry before use.


Thermometer contacts

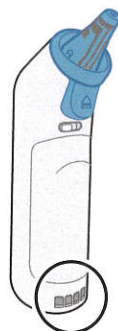


Approved cleaning solutions

- 70% isopropyl alcohol or ethyl alcohol

1

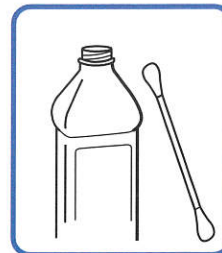

2

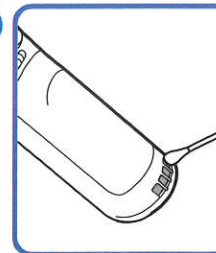

Place the thermometer aside for 1 minute,
allowing the contacts to air dry.


Lens window and probe



Approved cleaning solutions

- 70% isopropyl alcohol or ethyl alcohol only!


1

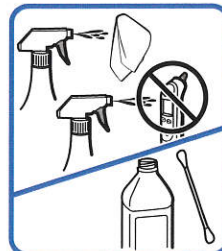

2



3

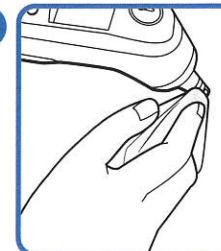

Wipe dry. Allow 5 minutes drying time.
Make sure probe lens window is dry before use.