



Digital Memo Board & Day Date Clock

SKU: 2511

Thank you for purchasing this Digital Memo Board, featuring a 10" electronic memo board with built in stand making it perfect for recording important reminders. The clear day, date clock helps support time orientation and reduce confusion, making this unit ideal for those living with Dementia and Alzheimer's. Please take the time to read this document thoroughly to familiarise yourself with how the product works and to ensure you get many years of safe use from this memo board and clock.

Operating Instructions

The digital memo board records notes in a monochrome format but simply drawing on the board. To prevent damage, it is recommended you use the PEN provided, this is clipped to the right of the board by magnet. The board can be erased by pressing CLEAR on the side of the unit, ensuring LOCK is in the lower position. To prevent notes being erased in error, the contents may be fixed by sliding LOCK to the upper position.



Figure 1

- 1) Clear
- 2) Lock
- 3) Pen

Rear of Unit:



Side of Unit:

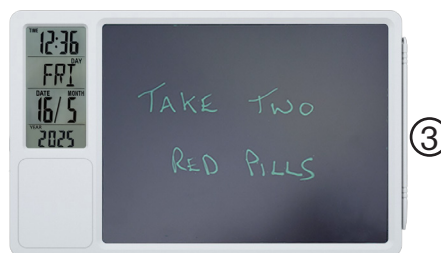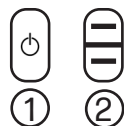

Setting the Clock

1. Press and hold MODE for 2 seconds to enter the clock setting mode - the year figure will flash.
2. Select the correct year by pressing UP or DOWN before moving to month by pressing MODE.
3. Repeat step 2 for the month, date, 12/24 hour mode (UP or DOWN switches between modes), hour and minute, the clock will then exit set mode.
4. The day of the week is displayed automatically, with AM or PM additionally being shown in 12 hour mode.

Setting the Alarm

1. While the clock shows the time, press MODE once to show the alarm time.
2. Press and hold MODE for 2 seconds to enter the alarm setting mode - the hour figure will flash.
3. Select the correct hour by pressing UP or DOWN before moving to minutes by pressing MODE.
4. Select the correct minute by pressing UP or DOWN, then press MODE to return to the current time.
5. Enable or disable the alarm by pressing UP. A bell between the hour and minute indicates the alarm is on.
6. The alarm will sound for 60 seconds but can be silenced by pressing any key.

Warranty (Please visit www.lifemaxuk.co.uk/warranty to register.)

In the unlikely event that your item develops a manufacturing fault, it is covered by a one year return to base warranty. Your warranty runs for 12 months from the date of purchase and is only valid with the original receipt, please ensure you keep this safe. Your warranty does not cover consumables, misuse or customer damage and will be deemed void where the repairs have been attempted by an unauthorised agent. This does not affect your statutory rights.

If your product develops a defect during the warranty period, please call the helpline number for further advice. If instructed, please securely pack your item and return enclosing your contact details, proof of purchase and fault details. We will, at our discretion, repair or exchange the item in line with warranty legislation. Where no proof of purchase is provided, or the product is outside the warranty period, repairs/replacements will be offered at cost to the customer.

Maintenance, Cleaning and Safety

1. This product is intended for indoor use only in a dry area free from humidity.
2. Remove batteries from the unit if it will not be used for an extended period, avoid letting them run flat
3. The unit may be cleaned with a soft, dry cloth. Avoid the use of cleaning products or solvent to clean.
4. This unit contains no user serviceable parts. Do not disassemble. Do not dispose of in a fire.
5. Keep this unit out of the reach of children and animals.



Lifemax Limited. 1 Pipers Court,
Thatcham, Berkshire. RG19 4ER. UK

Lifemax c/o Success Courier SL. Calle Rio
Tormes Num 1, Planta 1, Derecha, Oficina 3,
Fuenlabrada, Madrid. 28947. Spain

www.lifemaxuk.co.uk



2511_IM_251126