



# Soothing Sounds Pyramid

**SKU: 792**

## Instruction Manual



Thank you for purchasing a Lifemax Soothing Sounds Pyramid. We hope you'll get many years of use from this product. We recommend you read this document thoroughly to familiarise yourself with how your product works, how you should use it safely and effectively and how to look after your product to ensure it continues working as it should.



# Product Information

Relieve stress and combat the effects of tinnitus with the seven built in relaxing sounds and continual colour changing night light of this pyramid. Choice of heartbeat, babbling brook, forest wildlife, night-time sound, bird song, rain storm or ocean.


## What's included...?


Soothing Sounds Pyramid  
Instruction Manual

## How to use...

Slide open the cover on the base of the unit and insert 3 x AA batteries being careful to follow the polarity markings (+/-) on the battery and device.

Slide the left switch on the base to either "ON" or "30MINS". Please note the "30MINS" setting will only work in "SOUND" mode. Slide the right switch on the base to "LIGHT", "SOUND" or "L/S" for light and sound together.

Press the top of the pyramid to activate/deactivate device. If "SOUND" function was selected, press any of the three image buttons on the base of the unit to choose a desired sound. A further 4 sounds can be accessed by pressing the  button.

The volume can be changed by pressing the "Volume" button. There are three volume levels: Low, Medium and High. The sound / light & sound can be stopped at any time by pressing the .

After use, return the switch to the off position to preserve battery life.



Heartbeat



Babbling Brook



Forest Wildlife



Night-Time Sound



Birds Song



Rain Storm



Ocean

## How to look after...

To clean your device, wipe clean with a warm, damp cloth. Do not use harsh chemicals or abrasives to clean the device. Do not submerge the device in water.

## Safety Information

- This product requires 3 x AA size batteries (not included).
- Alkaline manganese batteries recommended.
- Slide off the battery door on the base of the unit.
- Remove the old batteries and insert new 3 x AA size batteries. Observe the correct polarity (+/-) when installing.
- Replace the battery door before use.
- Use only the designated battery type. Do not use re-chargeable batteries. Do not mix battery types.
- Read instructions carefully and install batteries correctly observing the polarity (+/-) sign to avoid leakage.
- Always replace sets of batteries at the same time. Do not mix old and new batteries.
- Do not attempt to recharge non-rechargeable batteries.
- Remove batteries from the product when it will not be used for an extended period of time to avoid leakage.
- Remove exhausted batteries to avoid leakage.
- Do not short out battery contacts.
- Do not dispose of batteries in fire.
- Do not immerse electrically operated products in water or other liquid.



# Warranty Information

Our Lifemax product is guaranteed for one year against defective parts or construction. This guarantee is from the date of purchase and is only valid with proof of purchase. Please note this guarantee does not cover normal wear and tear or damage caused by accident or maltreatment. The extended rights of this guarantee are in addition to the consumer statutory rights and do not limit the effect of these statutory rights.

Please register your warranty at <https://www.lifemaxuk.co.uk/warranty>.

# Technical Specification

<b>Model No</b>	792	<b>Power Output</b>	0.5W
<b>Power Input</b>	DC 3 x AA Batteries	<b>Amps</b>	0.1A
<b>Watts</b>	0.5W	<b>Lumen</b>	470
<b>LEDs</b>	1 x 5mm RGB		



This is a WEEE symbol. It indicates that when the end-user discards this product, it must be sent to separate collection facilities for recovery and recycling. By separating this product from other household-type waste, the volume of waste sent to incinerators or land-fills will be reduced and natural resources will thus be conserved. Lifemax Ltd provide an end-of-life solution for this product when sold in the UK by accepting back any end of life items for processing by a local registered waste disposal company.



Lifemax Limited, 1 Pipers Court,  
Thatcham, Berkshire, RG19 4ER

Lifemax (Ireland), 51 Bracken Road,  
Sandyford, Dublin, D18 CV48

[www.lifemaxuk.co.uk](http://www.lifemaxuk.co.uk)

